

QUICK START GUIDE



MILK GENIE

pumpables[™]



Power

Displays when plugged into power / charging



Memory Mode

Indicates a Program is Stored

Timer



Expression Mode

Indicates Expression Mode

Battery



Indicates Charge


Vacuum Level

1 - 10 Numbers & Bars

Power

Mode

Swap between Massage & Expression Modes

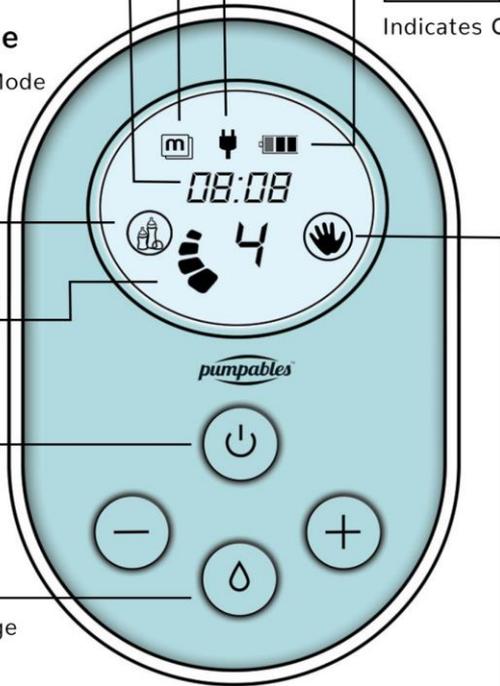

Press to Increase Vacuum

Hold 1 sec. to Increase Cycle Speed



Press to Decrease Vacuum

Hold 1 sec. to Decrease Cycle Speed




Massage Mode


Indicates Massage Mode

pumpables™

ASSEMBLY

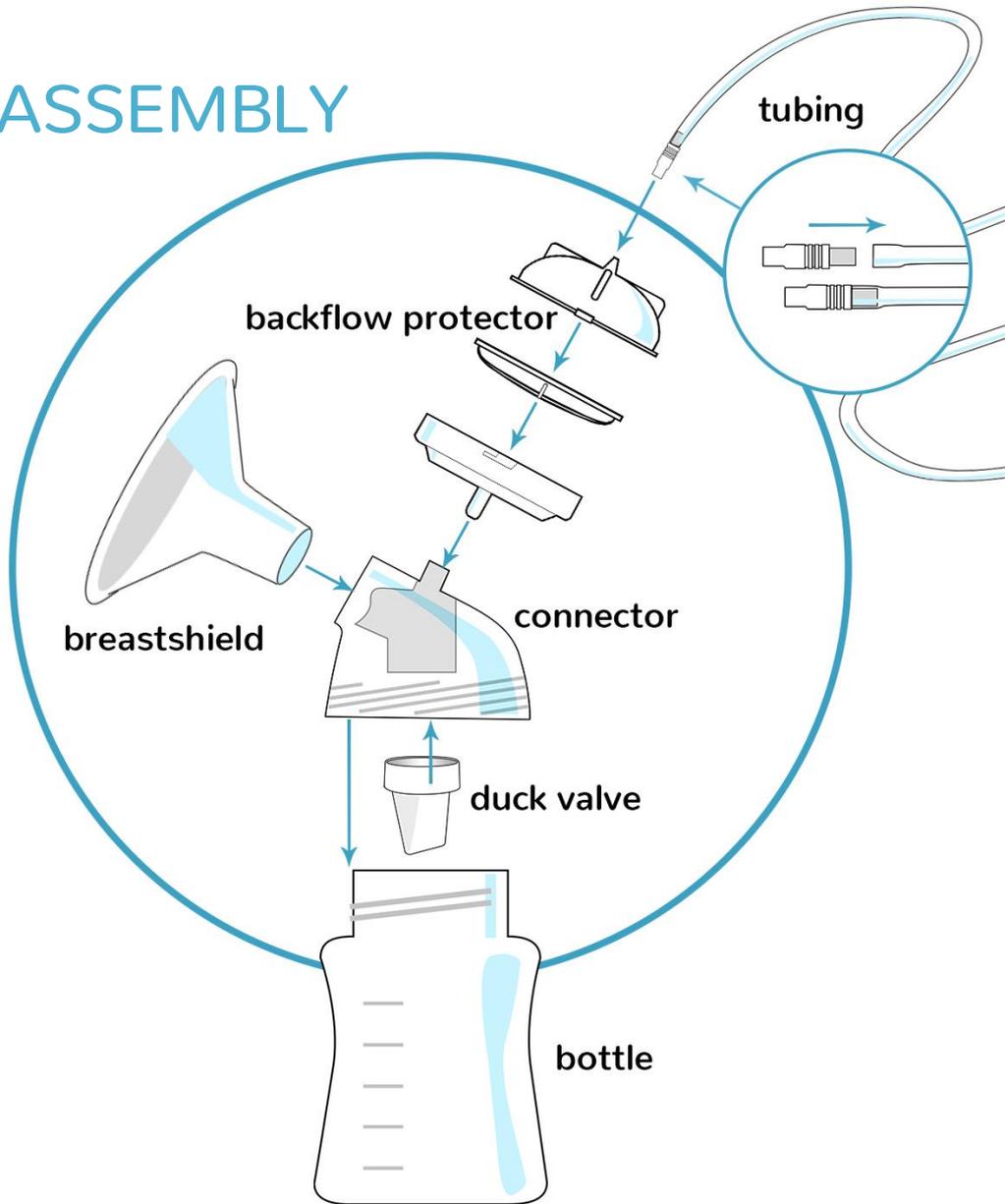


pumpables[™]

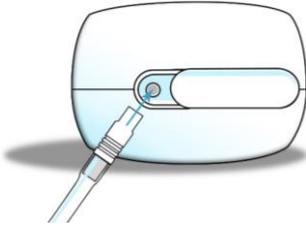

Single Pumping

Make sure unused airport is completely covered.

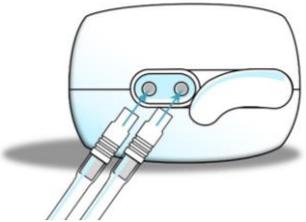


Double Pumping

Insert tubing into both air ports.

Assembling the Adapter & Charging

Slide in prong until you hear and feel a forceful click.

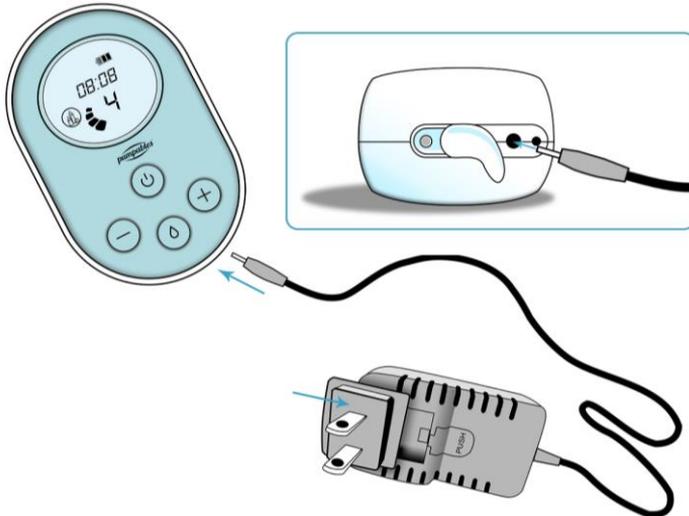

pumpables™

MEMORY MODE

1. Start Recording

Hold down power button.

2. Recording

 icon flashes while recording.

3. Stop Recording

Hold down power button.

 icon displays continuously when a program is saved.

Start Saved Program

Press and hold minus button.

Delete Saved Program

Press and hold mode button.


Stop Saved Program

Press and hold plus button.



pumpables™


BREASTSHIELD SIZING

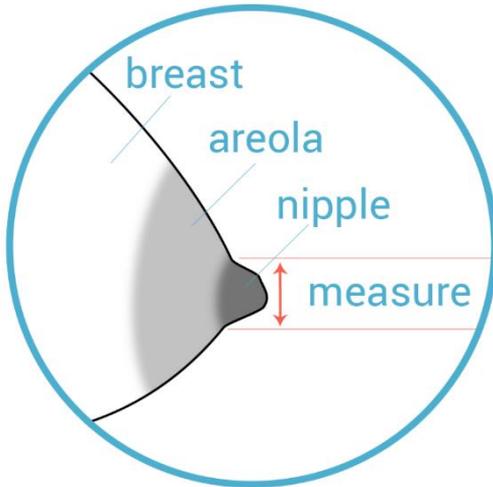

- Pump in Expression Mode for 5 Minutes
- Measure Diameter of Swollen Nipple
- Select Shield 2-3mm Wider than Nipple

Signs of a good fit are no pain, rubbing, redness, or whiteness of the nipple; and adequate milk expression.

You should not experience pain or discomfort while pumping. If so, try a different size or contact our customer support team for more help!

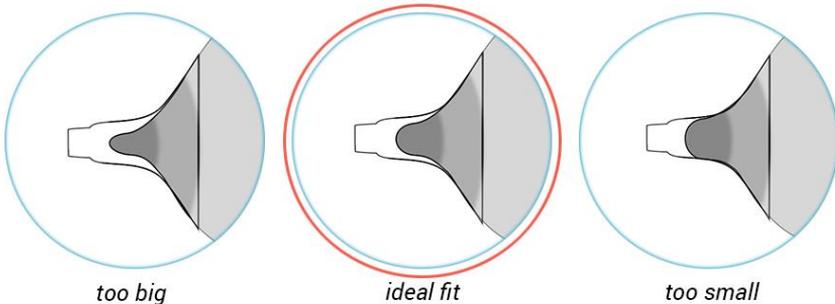

Complimentary nipple ruler printout & shield size consultation:
pumpables.co/measure | pumpables.co/fittingroom

pumpables[™]

NEED MORE HELP?

Our full manual is available at pumpables.co/manuals

Visit us at pumpables.co
or contact our expert customer service team!



facebook.com/pumpables



twitter.com/pumpables



hello@pumpables.co

We're pumping mamas too and we understand. Every one of our customer service team can assemble a breast pump blindfolded, and we love to help other pumping mamas.

pumpables[™]

© 2018 Pumpables Pte. Ltd.
ALL RIGHTS RESERVED

Milk Genie – ENG – QG – AUS – R00 - 2018.01

pumpables[™]